

Connect SW 5.1

UPGRADE DESCRIPTION

Connect SW 5.1 is the new software version for Primescan and Omnicam.

Installation and Requirements

- Connect SW 5.1 can be installed on all Primescan AC or Omnicam AC with the new cart and on the CEREC AC with Omnicam upgraded with the performance package
- For additional information about the performance package, please go to my.cerec.com
- Connect SW 5.1 is a full version and does not need any preinstalled versions
- For installation please close all applications, unpack the downloaded Connect SW 5.1 and the copy folder on your acquisition center. To start the installation, double-click the.exe and follow the installation setup

News and changes compared to Connect SW 5.0.1

- New selection for ordering aligners, splints or individual impression trays in the lab or in a central production
- Improved stability during acquisition phase
- Improved performance and stability during model calculation
- Integration of new TiBase*

Manufacturer	Tibase
DS	Ti Base: AstraTech OsseoSpeed TX AT TX 3.0 S; AT TX 3.5/4.0 L; AT TX 4.5/5.0 L
DS	Ti Base: XiVe XiVE 3.0 S; XiVE 3.4 S; XiVE 3.8 S; XiVE 4.5 L; XiVE 5.5 L

*Depending on the registration date of the medical device not all TiBase might be available in all countries at the same time. Please observe regional restrictions regarding TiBase and materials.