

Protocol of the CEREC treatment

Step by step towards success. Use this checklist to ensure that things will go smooth during your first treatment. Your steps are marked in dark gray, assistant tasks are marked in light gray.

✓	Task
☐	Turn on the CEREC device
☐	Set up the "Preparation" tray
☐	Set up the CEREC device (open the CEREC software, open/set up patient profile)
☐	Check static occlusion with red paper
☐	Scan the maxillary and mandibular quadrants/bite
☐	Anesthesia
☐	Preparation
☐	Gingiva management (retraction with thread or astringent paste; if needed, a hemostatic agent, such as Expasyl®)
☐	In case of using CEREC Omnicam, cut out the prepared tooth from the scan
☐	Postscanning of the preparation
☐	Manufacture of the restoration
☐	Shade selection
☐	Mounting the block
☐	Prepare "Insertion" tray
☐	Test the restoration
☐	Polish (proximal)
☐	Depending on the material, mounting and preparation of the ceramic part (etching, silanating)
☐	Tooth preparation (cleaning, etching, bonding)
☐	Keep tooth dry
☐	Insert
☐	Preliminary cleaning
☐	Press into final position and keep pressing
☐	Dental floss and final cleaning
☐	Remove thread, if applicable
☐	Harden
☐	Remove rubber dam if present
☐	Remove excess with the scalpel
☐	Occlusion check
☐	Adjustments
☐	Polish
☐	Fluoridate
☐	Save CEREC patient case
☐	Clean up room (remove instruments, clean/disinfect CEREC)